A QIAGEN / BD Company

PAXgene[®]
Blood RNA Tube

EN	English	3-8
BG	Български	9-14
CS	Česky	15-20
DA	Dansk	21-26
DE	Deutsch	27-32
EL	Ελληνικά	33-38
ES	Español	39-44
ET	Eesti	45-50
FI	Suomi	51-56
FR	Français	57-62
HR	Hrvatski	63-68
HU	Magyar	69-74
IS	Íslenska	75-80
IT	Italiano	81-86
KK	Қазақша	87-92
LT	Lietuvių k.	93-98
LV	Latviski	99-104
NL	Nederlands	105-110
NO	Norsk	111-116
PL	Polski	117-122
PT	Português	123-128
RO	Română	129-134
RU	Русский	135-140
SK	Slovensky	141-146
SL	Slovenski jezik	147-152
SV	Svenska	153-158
TR	Türkçe	159-164
UK	Українська	165-170
ZH	简体中文	171-176
	Symbol and Mark Key	177-178

EN - English

I. Intended Use

The PAXgene Blood RNA System consists of a blood collection tube (PAXgene Blood RNA Tube) and nucleic acid purification kit (PAXgene Blood RNA Kit). It is intended for the collection, storage, and transport of blood and stabilization of intracellular RNA in a closed tube and subsequent isolation and purification of intracellular RNA from whole blood for RT-PCR used in molecular diagnostic testing.

Performance characteristics for the PAXgene Blood RNA System have only been established with FOS and IL1B gene transcripts. The user is responsible for establishing appropriate PAXgene Blood RNA System performance characteristics for other target transcripts.

II. Summary and Explanation

The collection of whole blood is the first step in many molecular assays used to study intracellular RNA. A major challenge in this type of testing is the instability of intracellular RNA, which rapidly degrades within hours after blood collection. Furthermore, certain species of RNA, through the process of gene induction, increase in vitro after blood collection. Both in vitro RNA degradation and gene induction can lead to an under or overestimation of in vivo relative gene transcript number. The PAXgene Blood RNA Tube contains an additive that stabilizes the in vivo gene transcription profile by reducing in vitro RNA degradation and minimizing gene induction. When used in conjunction with the PAXgene Blood RNA Kit, the PAXgene Blood RNA Tube provides for accurate detection and quantitation of gene transcripts.

III. Specimen Collection and Preparation for Analysis

A. Required Blood Collection Accessories (Not included with the PAXgene Blood RNA Tube)

1. Blood collection set such as the BD Vacutainer® Push Button Blood Collection Set or the BD Vacutainer Safety-Lok™ Blood Collection Set. *See Ordering Information.*
2. A BD Vacutainer Needle Holder must be used to ensure proper function. *See Ordering Information.*
3. A "Discard Tube" if the PAXgene Blood RNA Tube is the only tube being drawn. *See Ordering Information.*
4. Labels for positive donor identification of samples.
5. Alcohol swab for cleansing site.
6. Dry sterile gauze.
7. Tourniquet.
8. Needle disposal container for used needle or needle/holder combination.

B. Procedure for Specimen Collection

1. Ensure that the PAXgene Blood RNA Tube is at 18–25°C prior to use and properly labeled with specimen identification.
2. If the PAXgene Blood RNA Tube is the only tube to be drawn, blood should be drawn into a "Discard Tube" prior to drawing blood into the PAXgene Blood RNA Tube so the interior volume of the blood collection set used during phlebotomy can be primed. Otherwise, the PAXgene Blood RNA Tube should be the last tube drawn in the phlebotomy procedure.

3. Using a blood collection set and a tube holder, collect blood into the PAXgene Blood RNA Tube using your institution's recommended procedure for standard venipuncture technique. **The following techniques shall be used to prevent possible backflow:**
 - a. Place donor's arm in a downward position.
 - b. Hold tube in a vertical position, below the donor's arm during blood collection.
 - c. Release tourniquet as soon as blood starts to flow into tube.
 - d. Make sure tube additives do not touch stopper or end of the needle during venipuncture.
4. **Allow at least 10 seconds for a complete blood draw to take place. Ensure that the blood has stopped flowing into the tube before removing the tube from the holder. The PAXgene Blood RNA Tube with its vacuum is designed to draw 2.5 ml of blood into the tube.**
5. Immediately after blood collection, gently invert the PAXgene Blood RNA Tube 8–10 times.
6. Store the PAXgene Blood RNA Tube upright at room temperature (18–25°C) for a minimum of 2 hours and a maximum of 72 hours before processing or transferring to refrigerator (2–8°C) or freezer (–20°C). See *Procedure for Freezing and Thawing Specimens in the PAXgene Blood RNA Tubes* for details if storage temperature of –70°C / –80°C is desired.

C. Procedure for Freezing and Thawing Specimens Collected in the PAXgene Blood RNA Tubes

1. Stand the PAXgene Blood RNA Tube upright in a wire rack. **Do not freeze tubes upright in a Styrofoam™ tray as this may cause the tubes to crack.**
2. The PAXgene Blood RNA Tubes can be stored at –20°C and below. If tubes are to be kept at temperatures below –20°C, freeze them first at –20°C for 24 hours, then transfer them to –70°C or –80°C.
3. Thaw the PAXgene Blood RNA Tubes in a wire rack at ambient temperature (18–25°C) for approximately two hours. **Do not thaw the PAXgene Blood RNA Tubes at temperatures above 25°C.**
4. Carefully invert the thawed PAXgene Blood RNA Tubes 10 times.

Note: If the two-hour ambient temperature incubation was not carried out prior to freezing, incubate tubes at least another two hours after reaching room temperature before processing.

Note: The frozen PAXgene Blood RNA Tubes are subject to breakage upon impact. To reduce the risk of breakage during shipment, frozen tubes should be treated in the same manner as glass tubes. Users must validate their own freezing and shipping protocol for the PAXgene Blood RNA Tubes.

D. Procedure for Specimen Preparation for Analysis

Blood samples must be processed according to the instructions provided with the PAXgene Blood RNA Kit.

For catalog numbers, see *Ordering Information*.

IV. Performance Characteristics

For a comprehensive listing of the performance characteristics for the PAXgene Blood RNA Tube, visit www.PreAnalytiX.com.

V. Limitations

1. **Under-filling of the PAXgene Blood RNA Tubes will result in an incorrect blood-to-additive ratio and may lead to incorrect analytic results or poor product performance.**
2. The PAXgene Blood RNA System is not suitable for the collection and purification of viral RNA.
3. Blood samples obtained with the PAXgene Blood RNA Tube must be prepared only with the PAXgene Blood RNA Kit. For catalog numbers for the PAXgene Blood RNA Kit, see *Ordering Information*.
4. The quantity of blood drawn should be 2.5 ml per PAXgene Blood RNA Tube, but this volume might vary with altitude, ambient temperature, barometric pressure, tube age, venous pressure, and filling technique.
5. The PAXgene Blood RNA Kit is intended for purification of intracellular RNA from human whole blood with leukocytes counts between 4.8–11.0 × 10⁶ leukocytes/ml.
6. Endotoxin not controlled. Blood and blood components collected and processed in the tube are not intended for infusion or introduction into the human body.


VI. Warnings and Precautions

- Contents of this tube are irritating to skin.
 - After inhalation, supply fresh air: consult doctor in case of complaints.
 - After skin contact, immediately wash with water and soap, and rinse thoroughly.
 - After eye contact, rinse opened eye for fifteen minutes under running water, then consult a doctor.
 - After swallowing, immediately call a doctor.
- A blood collection set must be used with the PAXgene Blood RNA Tube.** See *Ordering Information*.
- Practice Universal Precautions. Use gloves, gowns, eye protection, other personal protective equipment, and engineering controls to protect from blood splatter, blood leakage, and potential exposure to bloodborne pathogens.
- Handle all biologic samples and blood collection devices according to the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biologic samples (for example, through a needle stick injury), since they may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Utilize any built-in needle protector, if the blood collection set provides one. PreAnalytiX does not recommend resheathing used needles. However, the policies and procedures of your facility may differ and must always be followed.
- Discard all blood collection tubes in biohazard containers approved for their disposal.
- Do not re-use the PAXgene Blood RNA Tubes.
- Do not use the PAXgene Blood RNA Tubes after the expiration date printed on the tube label.
- Since the PAXgene Blood RNA Tube contains a chemical additive, a blood collection set must be used for blood collection to prevent possible backflow from the tube.
See *Specimen Collection and Preparation for Analysis* section.
- Excessive centrifugation speed (over 10,000 RCF) may cause PAXgene Blood RNA Tube breakage, exposure to blood and possible injury.
- Do not transfer a sample from a syringe to the PAXgene Blood RNA Tube.

VII. Storage

Store the unused PAXgene Blood RNA Tubes at 4–25°C. Limited excursion temperatures up to 40°C are permitted.

VIII. Instructions for Removal of BD Hemogard™ Closure

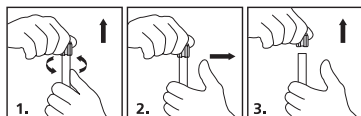- Grasp the PAXgene Blood RNA Tube with one hand, placing the thumb under the BD Hemogard Closure. (For added stability, place arm on solid surface). With the other hand, twist the BD Hemogard Closure while simultaneously pushing up with the thumb of the other hand **ONLY UNTIL THE TUBE STOPPER IS LOOSENED**.
- Move thumb away before lifting closure. **DO NOT** use thumb to push closure off tube. Caution: If the tube contains blood, an exposure hazard exists. To help prevent injury during closure removal, it is important that the thumb used to push upward on the closure is removed from contact with the tube as soon as the BD Hemogard Closure is loosened.
- Lift closure off tube. In the unlikely event of the plastic shield separating from the rubber stopper, **DO NOT REASSEMBLE CLOSURE**. Carefully remove rubber stopper from tube.

IX. Technical Assistance

If you have any questions regarding the PAXgene Blood RNA System, contact one of the BD Technical Services Departments listed on BD – Customer Service section.

X. Ordering Information**PAXgene Products****PAXgene Blood RNA Tube / 762165**

Content: 100 Blood collection tubes. Use with the PAXgene Blood RNA Kit (50)

2.5 ml • 16 × 100 mm	6.9 ml additive	Sterile
Plus • BD Hemogard	100 tubes/case	IVD

PAXgene Blood RNA Kit (50) 762164 N.A. / 762174 Ex. N.A.

Content: 50 PAXgene Spin Columns, Processing Tubes, RNase-free Reagents and Buffers.

Use with the PAXgene Blood RNA Tubes (100)

BD Products***BD Vacutainer Push Button Blood Collection Set / 367344**

Content: 21G 3/4 inch (0.8 × 19 mm) needle, 12 inch (305 mm) tubing with luer adapter. 50/box, 200/case

BD Vacutainer Safety-Lok Blood Collection Set / 367281 US / 367286 CE

Content: 21G 3/4 inch (0.8 × 19 mm) needle, 12 inch (305 mm) tubing with luer adapter. 50/box, 200/case

BD Vacutainer One Use Holder / 364815

Content: Case only for 13 mm and 16 mm diameter. 1000/case

BD Vacutainer Plus Serum Tubes / 367812 US / 368975 CE

Content: 13 × 75 mm 4.0 ml draw with red BD Hemogard closure and paper label. 100/box, 1000/case

* These catalog numbers represent typical products that can be used with the PAXgene Blood RNA Tube.

To order BD Blood Collection Accessories:

Go to: <http://www.bd.com/support/contact/international.asp>

BD - Customer Service**Argentina, Uruguay and Paraguay**

Orders: 0800.444.5523
E-mail: crc_argentina@bd.com

Australia

Orders: 1.800.656.100
Fax: 1.800.656.110
E-mail: bd_anz@bd.com

Austria

Orders: 43.1.706.36.60
Fax: 43.1.706.36.60-11
Email: customerservice.vac.ga@europe.bd.com

Belgium

Orders: 32.53.720.556
Fax: 32.53.720.549
Orders: orders.be@europe.bd.com

Brazil

Orders: 0800.055.56.54
E-mail: consultoria_vacutainer@bd.com

Canada

Orders: 800.268.5430
Fax: 800.565.0897
E-mail: custsvccan@bd.com

Central and Eastern Europe (CEE)

Orders: 48.22.377.11.11
Fax: 48.22.377.11.02
Bulgaria: info_bulgaria@europe.bd.com
Czech Republic: info_czech@europe.bd.com
Croatia: info_croatia@europe.bd.com
Hungary: info_hungary@europe.bd.com
Poland: info_poland@europe.bd.com
Romania: info_romania@europe.bd.com
SEE: info_balkan@europe.bd.com
Serbia: info_serbia@europe.bd.com
Slovakia: info_slovakia@europe.bd.com
Slovenia: info_slovenia@europe.bd.com
Technical: bd_service_poland@bd.com

Denmark

Orders: 45.43.43.45.66
Fax: 45.43.96.56.76
Orders: ordre.dk@europe.bd.com
Technical: bddenmark@europe.bd.com

Finland

Orders: 358.9.88.70.780
Fax: 358.9.88.70.7816
Orders: tilaukset.fi@europe.bd.com
Technical: bdsuomi@europe.bd.com

France

Orders: 33.476.68.94.96
Fax: 33.476.68.36.93
Orders: commandesfr@europe.bd.com
E-mail: serviceclientbdf@europe.bd.com
Technical: vacutainerfr@bd.com

Germany

Orders: 49.6221.305.553
Fax: 49.6221.305.379
Email: customerservice.vac.ga@europe.bd.com

Italy

Orders: 39.02.48.240.500
Technical: 39.335.424.388
Fax: 39.02.40.91.80.11
E-mail: ordini.it@europe.bd.com

Middle East & Africa (EMA)

Orders: 971.4.3379525
Fax: 971.4.03379551
E-mail: EMA_PAS@bd.com

The Netherlands

Orders: 31.20.582.94.20
Fax: 31.20.582.94.21
Orders: orders.nl@europe.bd.com

New Zealand

Orders: 0800.572.468
Fax: 0800.572.469
E-mail: nz_customerservice@bd.com

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Orders: 47.23.32.30.00
Fax: 47.23.32.30.99
E-mail: kundeservice@puls-norge.no

Spain, Portugal and Andorra

Orders: 34.91.848.8104
Technical: 34.902.27.17.27
Fax: 34.91.828.81.48
E-mail: info.spain@europe.bd.com

Sweden

Orders: 46.8.775.51.60
Fax: 46.8.775.51.95
Orders: order.se@europe.bd.com
Technical: bdsSweden@europe.bd.com

Switzerland

Orders: 41.61.485.22.22
Fax: 41.61.485.22.00
E-mail: infoch@europe.bd.com

UK

Orders: 44.1.865.781.666
Fax: 44.1.865.717.313
E-mail: bduk_customerservice@europe.bd.com

USA

Orders: 888.237.2762
Technical: 800.631.0174
Fax: 800.847.2220
Technical: www.bd.com/vacutainer/contact

QIAGEN – Customer Service**Australia**

Orders: 1.800.243.800
 Technical: 1.800.243.066
 Fax: 03.9840.9888

Austria

Orders: 0800.28.10.10
 Technical: 0800.28.10.11
 Fax: 0800.28.10.19

Belgium

Orders: 0800.79612
 Technical: 0800.79556
 Fax: 0800.79611

Brazil

Orders: 0800.557779
 Technical: 0800.557779
 Fax: 55.11.5079.4001

Canada

Orders: 800.572.9613
 Technical: 800.DNA.PREP (800.362.7737)
 Fax: 800.713.5951

China

Orders: 86.21.3865.3865
 Technical: 800.988.0325
 Fax: 86.21.3865.3965

Denmark

Orders: 80.885945
 Technical: 80.885942
 Fax: 80.885944

Finland

Orders: 0800.914416
 Technical: 0800.914413
 Fax: 0800.914415

France

Orders: 01.60.920.926
 Technical: 01.60.920.930
 Fax: 01.60.920.925

Germany

Orders: 02103.29.12000
 Technical: 02103.29.12400
 Fax: 02103.29.22000

Hong Kong

Orders: 800.933.965
 Technical: 800.930.425
 Fax: 800.930.439

Ireland

Orders: 1800.555.049
 Technical: 1800.555.061
 Fax: 1800.555.048

Italy

Orders: 800.789.544
 Technical: 800.787.980
 Fax: 02.334304.826

Japan

Orders: 03.6890.7300
 Technical: 03.6890.7300
 Fax: 03.5547.0818

Korea (South)

Orders: 080.000.7146
 Technical: 080.000.7145
 Fax: 02.2626.5703

Luxembourg

Orders: 8002.2076
 Technical: 8002.2067
 Fax: 8002.2073

Mexico

Orders: 01.800.7742.639
 Technical: 01.800.7742.436
 Fax: 01.800.1122.330

The Netherlands

Orders: 0800.0229592
 Technical: 0800.0229602
 Fax: 0800.0229593

Norway

Orders: 800.18859
 Technical: 800.18712
 Fax: 800.18817

Singapore

Orders: 1800.742.4362
 Technical: 1800.742.4368
 Fax: 65.6854.8184

Spain

Orders: 91.630.7050
 Technical: 91.630.7050
 Fax: 91.630.5145

Sweden

Orders: 020.790282
 Technical: 020.798328
 Fax: 020.790582

Switzerland

Orders: 055.254.22.11
 Technical: 055.254.22.12
 Fax: 055.254.22.13

UK

Orders: 01293.422.911
 Technical: 01293.422.999
 Fax: 01293.422.922

USA

Orders: 800.426.8157
 Technical: 800.DNA.PREP (800.362.7737)
 Fax: 800.718.2056

The PAXgene Blood RNA Tube is made in the UK by BD for PreAnalytiX GmbH.

Patent www.PreAnalytiX.com/patents

PAXgene and PreAnalytiX are trademarks of PreAnalytiX GmbH; QIAGEN is a trademark of QIAGEN NV; Styrofoam is a trademark of The Dow Chemical Company; all other trademarks are property of Becton, Dickinson and Company.

 PreAnalytiX GmbH, Feldbachstrasse, 8634 Hombrechtikon, CH ©2019 PreAnalytiX GmbH www.PreAnalytiX.com

Australian Sponsor:

Becton Dickinson Pty Ltd.
 4 Research Park Drive
 North Ryde, NSW 2113
 Australia

New Zealand Sponsor:

Becton Dickinson Limited
 14b George Bourke Drive
 Mt. Wellington, Auckland 1060
 New Zealand

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Conformite European

In Vitro diagnostic medical device

Do not Reuse

Use By

Method of Sterilization Using Irradiation

Contains sufficient for <n> tests

Manufacturer

Batch Code

Temperature Limitation

Catalog Number

EN This end up
BG Тази страна отгоре
CS Touto stranou nahoru
DA Denne side op
DE Diese Seite nach oben
EL Αυτή η πλευρά προς τα πάνω
ES Este lado hacia arriba
ET See pool üles
FI Tämä puoli ylöspäin
FR Haut
HR Ovaj završetak prema gore
HU Ez a vég felül
IS Þessi endi upp
IT Questa estremità in alto
KK Соңы
LT Šis galas viršuje
LV Augšgals
NL Deze zijde boven
NO Denne side op
PT Ta strana do góry
PL Este lado para cima
RO Partea aceasta în sus
RU Верх
SK Touto stranou nahor
SL Ta stran navzgor
SV Denna sida upp
TR Bu Uc Yukarı
UK Догори
ZH 此端向上

EN Fragile
BG Чупливо, Използвайте внимателно
CS Křehké, manipulovat opatrně
DA Forsigtig
DE Zerbrechlich, mit Vorsicht zu handhaben
EL Ευθραστο, χειριστείτε με προσοχή
ES Frágil, manipular con cuidado
ET Habras, käsitseda ettevaatlikult
FI Särkyvä, käsittele varoen
FR Fragile, à manipuler avec précaution
HR Lomljivo, rukujte s oprezom
HU Törékeny, Óvatosan bánjon a termékkel
IS Brothætt, meðhöndlið með gát
IT Fragile, Maneggiare con cura
KK Морт сынғыш, абайлап қолданыңыз
LV Trausls, lietot uzmanīgi
NL Voorzichtig: breekbaar
NO Forsiktig
PL Ostrożnie, szkło
PT Frágil, manusear com cuidado
RO Fragil, Manevrați cu grijă
RU Хрупкое, обращаться с осторожностью
SK Krehké, manipulujte opatrne
SL Lomljivo, ravnaj previdno
SV Ömtåligt, hanteras varsamt
TR Hassas, Dikkatlice Ele Alınız
UK Крихке, поводитись з обережністю
ZH 易碎, 小心轻放

EN Recyclable USA
BG Подлежи на рециклиране в САЩ
CS Recyklovatelné USA
DA Eget til genindvinding i USA
DE Recyclingfähig USA
EL Ανακυκλώσιμο ΗΠΑ
ES Reciclable en EE.UU.
ET Ringlussevõetav, USA
FI Yhdysvaltoja koskevat kierrätystiedot
FR Recyclable aux États-Unis
HR Za reciklažu SAD
HU Újrahasznosítható USA
IS Endurvinnanlegt BNA
IT Riciclabile USA
KK Ҷайта өңделетін АҚШ
LT Perdirbama JAV
LV Derīgs atreciņšējai pārstrādei ASV
NL Recyclebaar VS
NO Eget til genindvinding i USA
PL Produkt nadaje się do przetworzenia USA
PT Reciclável EUA
RO Reciclabil SUA
RU Пригодно для переработки США
SK Recyklačia USA
SL Za reciklažo v ZDA
SV Återvinningsbart i USA
TR Geridönümlü ABD
UK Придатний для переробки в США
ZH 在美国可回收

H411

EN Toxic to aquatic life with long lasting effects.
BG токсичен за водните организми, с дълготрайни поражения.
CS Toxický pro vodní organismy, s dlouhodobými účinky.
DA Giftig for vandlevende organismer, med langvarige virkninger.
DE Giftig für Wasserorganismen, mit langfristiger Wirkung.
EL Τοξικό για τους υδρόβιους οργανισμούς, με μακροχρόνιες επιπτώσεις.
ES Tóxico para los organismos acuáticos, con efectos nocivos duraderos.
ET Mürgine veeorganismidele, pikaajaline toime.
FI Myrkyllinen vesirorganismeille, aiheuttaa pitkäkestoisia vaikutuksia.
FR Toxique pour les organismes aquatiques, entraîne des effets néfastes à long terme.
HR Otrovno za vodeni okoliš s dugotrajnim učincima.
HU Vízi szervezetek számára mérgező, hosszú távú következményekkel.
IS Eitrað lífi í vatni, hefur langvinn áhrif.
IT Altamente tossico per gli organismi acquatici con effetti di lunga durata.
KK Судагы ағзаларға улы және әсері ұзаққа созылады.

LT Toksiška vandens organizmams, sukelia ilgalaikius pakitimus.
LV oksisks ūdens organismiem ar ilgstošām sekām.
NL Giftig voor in het water levende organismen, met langdurige gevolgen.
NO Giftig, med langtidsvirkning, for liv i vann.
PL Działal toksyczna na organizmy wodne, powodując długotrwałe skutki.
PT Tóxico para os organismos aquáticos com efeitos duradouros.
RO Toxic pentru mediul acvatic cu efecte pe termen lung.
RU Токсично для водных организмов с долгосрочными последствиями.
SK Toxické pre vodné formy života s dlhotrvajúcimi efektmi.
SL Strupeno za vodne organizme, z dolgotrajnimi učinki.
SV Giftigt för vattenlevande organismer med långtidseffekter.
TR Uzun süreli etkilerle sudaki yaşam için zehirlidir.
UK токсичний для водних організмів із довготривалими наслідками.
ZH 对水生生物有毒, 可产生长期不良影响。

H315 - H317

EN DANGER: H315 Causes skin irritation H317 May cause an allergic skin reaction	IS HÆTTA: H315 Veldur húðberingu H317 Getur valdið ofnæmsviðbrögðum í húð	SK NEBEZPEČENSTVO: H315 Spôsobuje podráždenie pokožky H317 Môže spôsobiť alergickú reakciu pokožky
BG ОПАСНОСТ: H315 Причинява дразнене на кожата H317 Може да причини алергична кожна реакция	IT PERICOLO: H315 Provoca irritazione cutanea H317 Può provocare una reazione allergica cutanea H319 Provoca grave irritazione oculare	SL NEVARNOST: H315 Povzroča draženje kože H317 Lahko povzroči alergijski odziv kože
CS NEBEZPEČÍ: H315 Dráždí kůži H317 Může vyvolat alergickou kožní reakci	KK ҚАУІПТІ: H315 Теріні тітіркендіреді H317 Теріде аллергиялық реакция тудыруы мүмкін	SV FARA: H315 Irriterar huden H317 Kan orsaka allergisk hudreaktion
DA FARE: H315 Forårsager hudirritation H317 Kan forårsage allergisk hudreaktion	LT PAVOJUS: H315 Dirgina od H317 Gali sukelti alerginę odos reakciją	TR TEHLİKE: H315 Cilt tahrişine yol açar H317 Alerjik cilt reaksiyonlarına yol açar
DE GEFÄHR: H315 Hautreizend H317 Kann allergische Hautreaktionen auslösen	LV BĪSTAMI: H315 kairina ādu H317 var izraisīt alerģisku ādas reakciju	UK НЕБЕЗПЕКА: H315 Подразнює шкіру H317 Може спричинити алергічну шкірну реакцію
EL ΚΙΝΔΥΝΟΣ: H315 Προκαλεί ερεθισμό του δέρματος H317 Μπορεί να προκαλέσει αλλεργική δερματική αντίδραση	NL GEVAAR: H315 Veroorzaakt huidirritatie H317 Kan een allergische reactie huidreactie veroorzaken	ZH 危险: H315 造成皮肤刺激。 H317 可能引起皮肤过敏反应。
ES PELIGRO: H315 Provoca irritación cutánea H317 Puede provocar una reacción alérgica en la piel.	NO FARE: H315 Forårsager hudirritation H317 Kan forårsage allergisk hudreaktion	
ET OHT: H315 Põhjustab nahaärritust H317 Võib põhjustada allergilist nahareaktsiooni	PL NIEBEZPIECZENSTWO: H315 Działa drażniąco na skórę H317 Może powodować reakcję alergiczną skóry	
FI VAARA: H315 Ihoa ärsyttävä H317 Voi aiheuttaa allergisen ihoreaktion	PT PERIGO: H315 Provoca irritação na pele H317 Pode provocar uma reacção alérgica cutânea	
FR DANGER: H315 Provoque une irritation cutanée H317 Peut provoquer une réaction allergique cutanée	RO PERICOL: H315 Cauzează iritații cutanate H317 Poate cauza reacții alergice cutanate	
HR OPASNOST: H315 Nadražuje kožu H317 Može prouzročiti alergijsku reakciju na koži	RU ОПАСНО: H315 Вызывает раздражение кожи H317 Может вызвать аллергическую реакцию на коже	
HU VIGYÁZAT! H315 Bőrirritációt okoz H317 Allergiás bőrreakciót válthat ki		

H318

EN Causes serious eye damage	LT Smarkiai pažeidžia akis.
BG причинява сериозно увреждане на очите.	LV izraisa nopietnus acu bojājumus.
CS Způsobuje vážné poškození očí.	NL Veroorzaakt ernstig oogletsel.
DA Forårsager alvorlig øjenskade.	NO Gir alvorlig øyeskade.
DE Verursacht schwere Augenschäden.	PL Powoduje poważne uszkodzenie oczu.
EL Προκαλεί σοβαρή οφθαλμική βλάβη.	PT Provoca lesões oculares graves.
ES Provoca lesiones oculares graves.	RO Provoacă leziuni oculare severe.
ET Põhjustab raskeid silmakahjustusi.	RU Вызывает серьезные повреждения глаз.
FI Aiheuttaa vakavia silmävaurioita.	SK Spôsobuje vážne poškodenie očí.
FR Provoque des lésions oculaires graves.	SL Povzroča hude poškodbe oči.
HR Uzrokuje teške ozljede oka.	SV Orsakar allvarliga ögonskador.
HU Súlyos szemsérülést okoz.	TR Ciddi göz hasarına neden olur.
IS Veldur alvarlegum augnskaða.	UK призводить до тяжкого ураження очей.
IT Causa gravi lesioni oculari.	ZH 造成严重眼部损伤。
KK Көзге ауыр зақым келтіреді.	

BD - Preanalytical Systems • Franklin Lakes, NJ

REV LEVEL	ALOA #	ECR/O #	DESCRIPTION	DESIGNER	DATE	SAP #
01	01066-02	ECR/O 5-146965	First Issue. VDP40400 Web product insert for VD1750040 paper product insert. Remove print information. Include Australian and New Zealand address. Remove SAP number. Update copy right through out the document. Section VII change temp limit from 18-25C to 4-25C. Include Patent www.PreAnalytiX.com/patents. Include IVD symbol for all translations.	N. Tattersall	10.July.2019	N/A
PACKAGING LEVEL			Product Circular - ONLY WEB (must match the content of VD1750040 paper IFU)	DRAWING #	VDP40400-01	

PMS 023 C

PMS 3272 C

PMS 287 C

Black